

Oral Microbiome

Test Instructions


Warning

- For external use only.
- Do not drink, touch or remove the reagent from the vial.
- Small cap poses a choking hazard.
- Do not use if the product is visibly damaged.

Important Information

- If the liquid gets in your eyes or on your skin, wash the affected area immediately and seek medical advice.
- Avoid consuming food, beverages, smoking, chewing gum, or brushing your teeth for at least 30 minutes before collecting the sample.

Test Kit Components

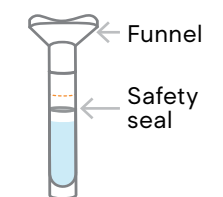

Collection Tube



Tube Cap



Safety Bag
with Pad



Return Box



Return Envelope
Mailing Label

> 1 Registering your sample




- 1.1 Scan the QR code or visit regeneruslabs.salient.bio
- 1.2 Enter your sample barcode and click **Next** (You only need to register the test once)
- 1.3 Fill in the required fields and click **Register my barcode**

Your sample barcode


> 2 Sample collection

2.1

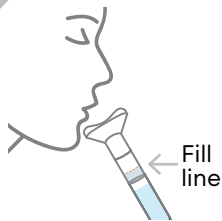


Place funnel opening under your mouth and spit saliva into funnel. Repeat until saliva liquid (not foam) reaches the fill line.

2.2

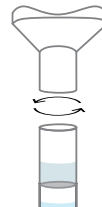

Hold the tube and twist to remove the funnel. Discard the funnel.

2.3

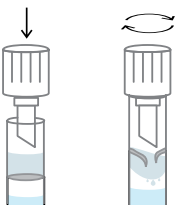

To close the tube, hold it upright and press down while twisting the cap until it is tightly secured.


2.4



Shake the closed tube vigorously a few times.


> 3 Returning your sample

3.1

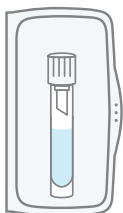

Place the Collection Tube into the Return Box. Close the box, and make sure you hear the click sound.

3.2

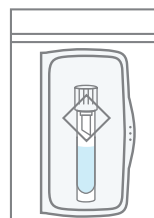

Place the Return Box with the Sample into the Safety Bag and seal the Safety Bag.

3.3



Insert the packed sample into the Return Envelope. Attach the Return Label and follow the returns instruction included in your test kit.

Any Issues?

Email info@regeneruslabs.com describing the problem along with photos of kit components related to the issue.